


# Filtration, Identification and Applications of CTCs and Circulating Cancer-Associated Cells

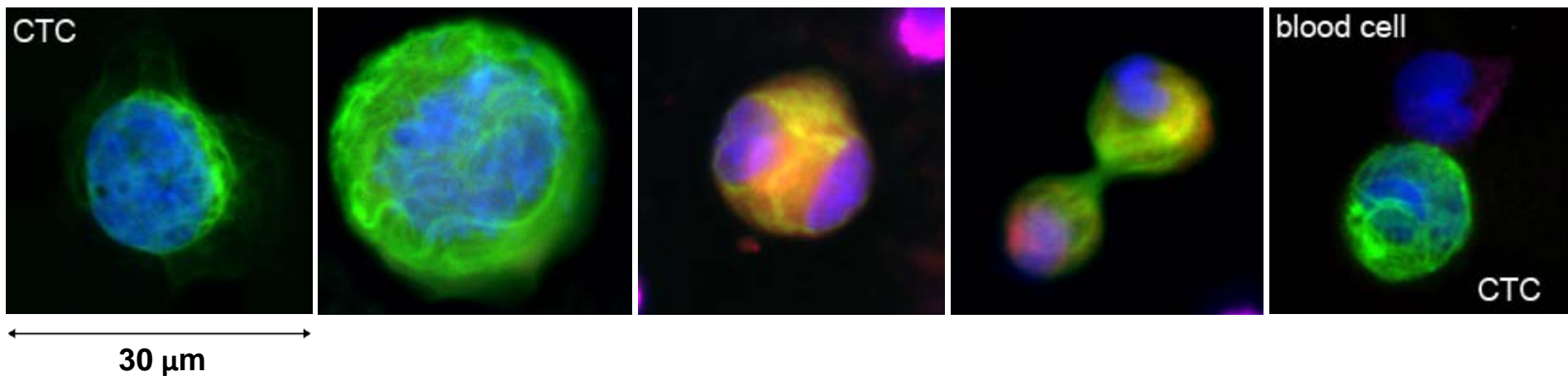

Cha-Mei Tang, Dan Adams, and Peixuan Zhu  
Creatv MicroTech, Inc.


February 10, 2014

- CellSieve™ microfilter
- CTC
- Circulating giant cancer-associated cells

# CellSieve™ Microfilter


**Independent of CTC surface markers**

SEM Image

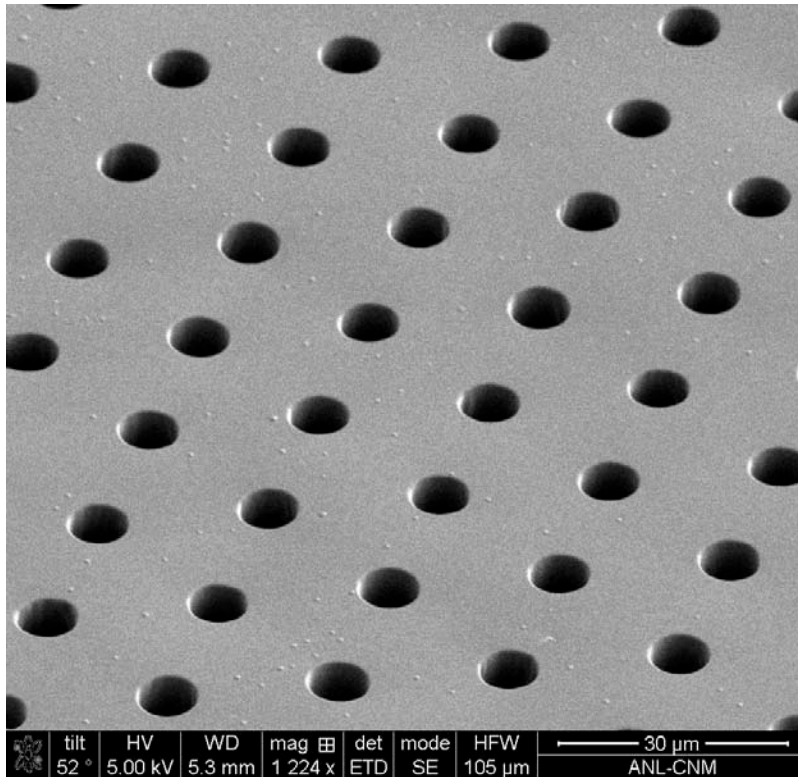

- Uniform 7µm pore size and distribution
- Non-fluorescent
- Lay flat on glass slide

# Filtration & Assay System

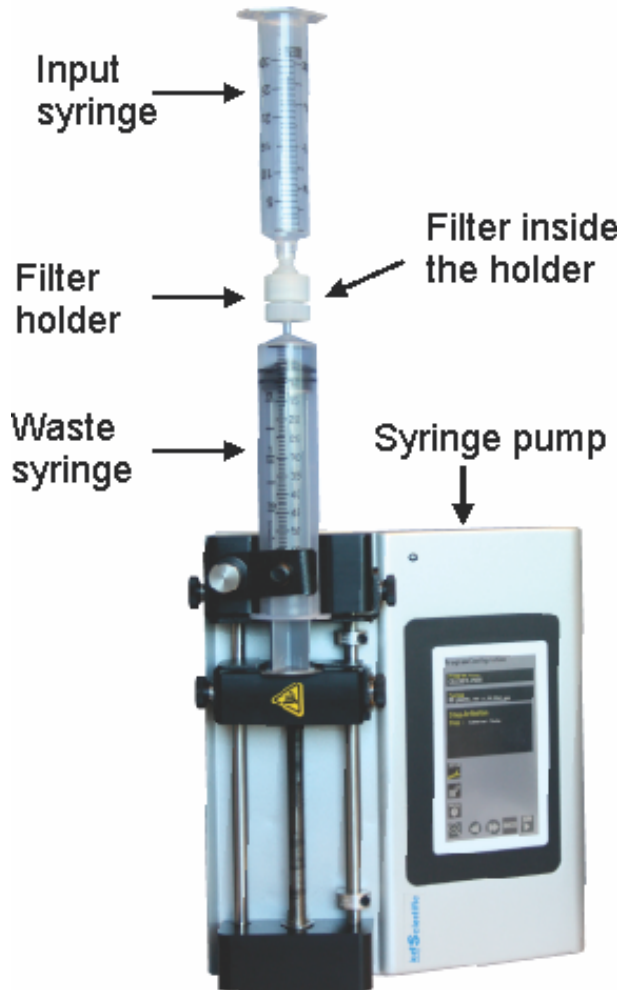


- Rapid
  - 3 min to filter 7.5 ml of blood
- Straightforward work flow
  - Entire enumeration assay can be performed in the filter holder
- Small and large sample size
- Consistent flow
- Gentle - no cell damage
- Low cost instrument

- Assays of cells on the filter can be performed in the holder
- Cells can be back washed out
- **Single** cells can be picked off filter
- Directly lysis of cells on filter
- Sample types
  - Blood
  - Bone marrow
  - Urine
  - CSF

## “True” CTCs

Criteria to improve specificity

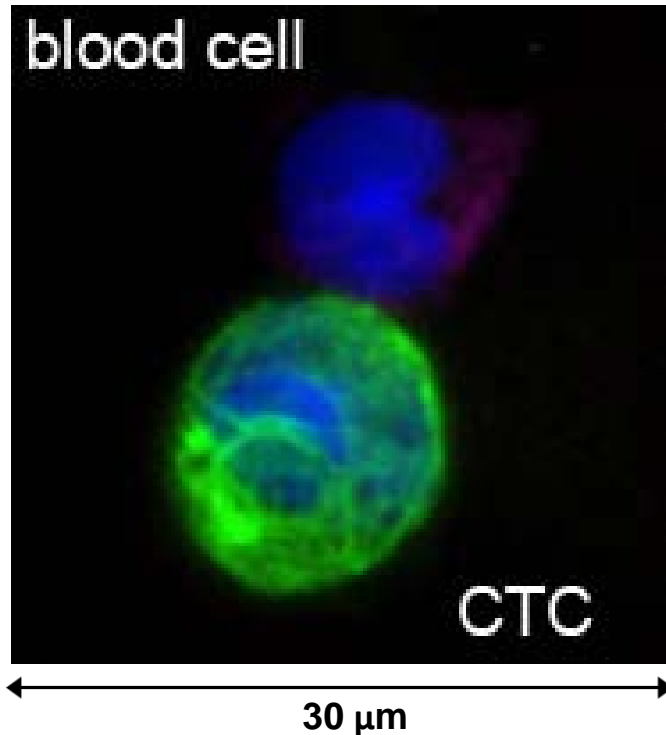


- DAPI positive
  - and cancerous looking
- To identify CTC
  - CK 8, 18, 19 (+)
  - and filamentous pattern
- To eliminate blood cells
  - CD45 (-)

Cancers of mesenchymal origin require other markers.

# Circulating Cancer Associated Macrophage-like Cells (CAMLs)

Prostate

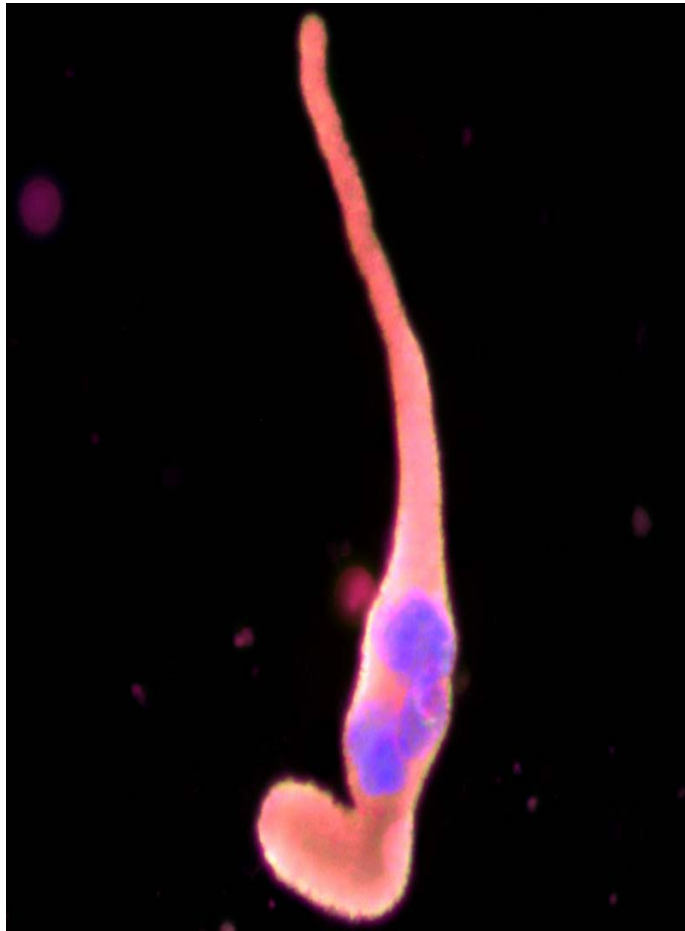


128 μm

- Large atypical nucleus
- May express CK 8, 18 or 19 and EpCAM **and**  
CK and EpCAM are diffused
- Most of time CD45 positive
- Large: 20 - 300 μm
- Express CD11c, CD14
- Express endothelial markers CD146, TIE-2

**“In press” open access PNAS, under embargo**

# Breast Cancer Patient

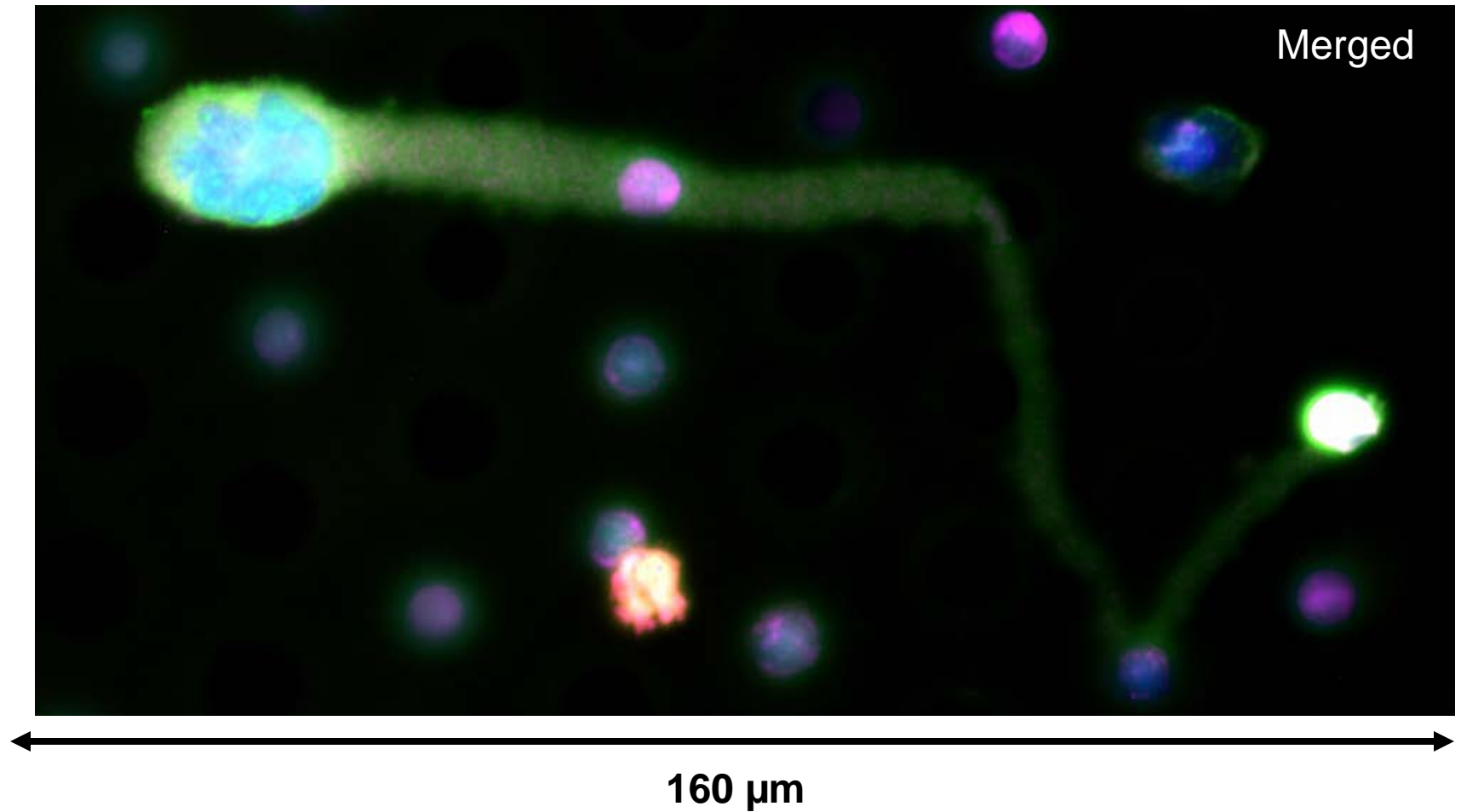



# Interactions of CAMLs with CTCs


Merged

DAPI

CK 8, 18, 19

EpCAM

CD45

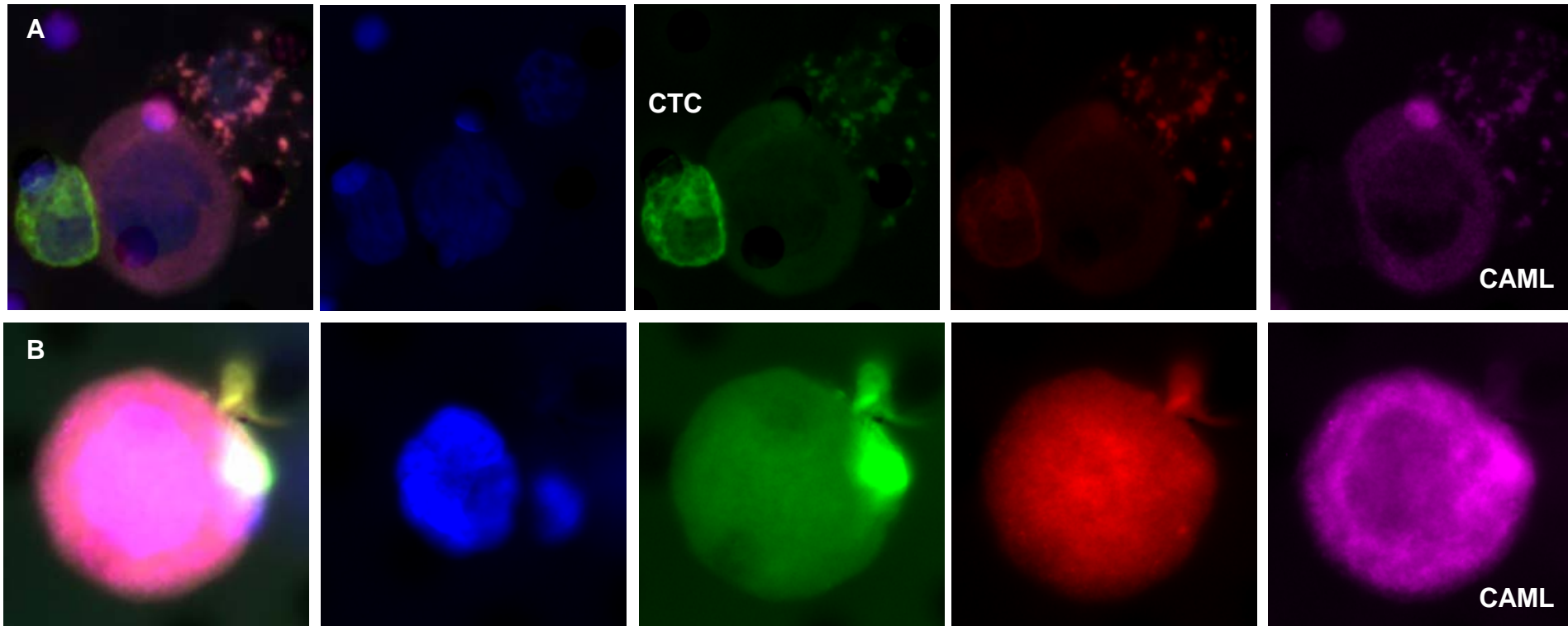


- CAMLs engulfing CTCs – explain presence of marks in CAMLs
- Found in ~10% of patient samples
- Short survival

# CAMLs with Ingested Markers


## Prostate CAML

Merge

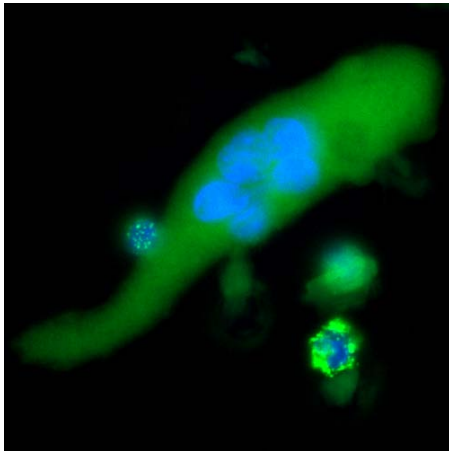


80 μm

PSMA

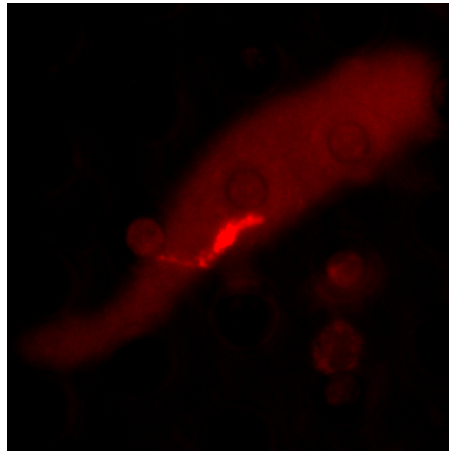


## Pancreatic CAML

Merge

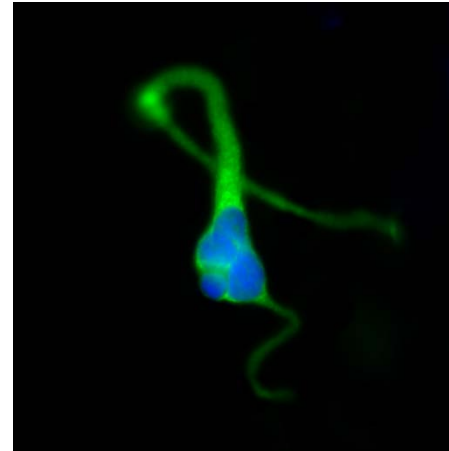


PDX-1

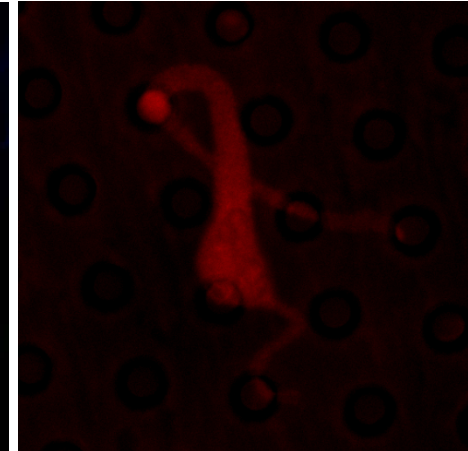

# CAMLs More Prevalent than CTCs

## Found even in stage I/II patients

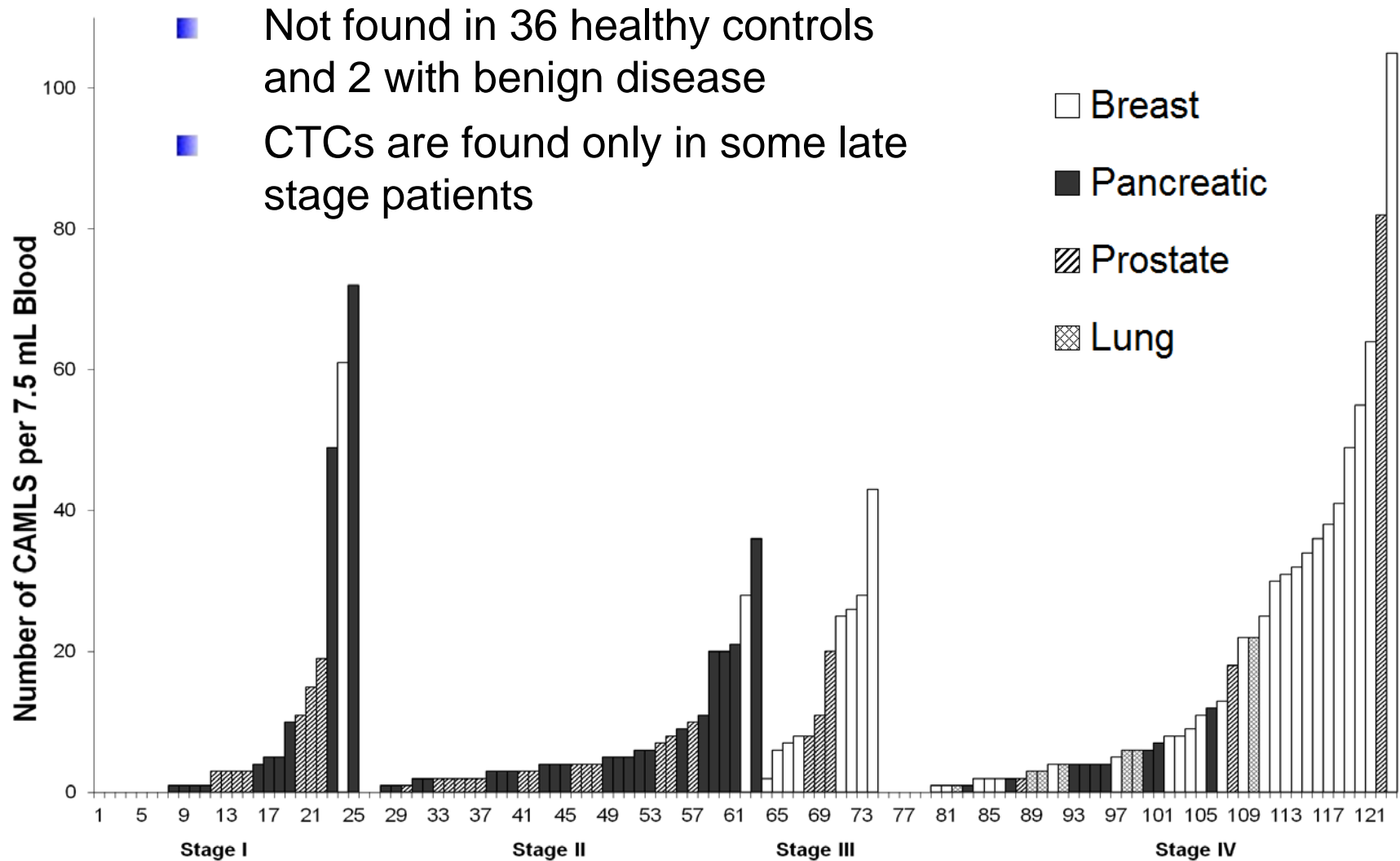


# Clinical Utility of CAMLs

- Important to differentiate CTCs and CAMLs
  - They have different respond to therapy
- CAMLs more prevalent than CTCs
- May provide rapid indication of effectiveness of therapy together with CTCs
- Predict survival
- Potential biomarker for screening at risk patients
  - Lung cancer
  - Pancreatic cancer

# Summary

- CellSieve™ microfilter
  - Applicable to all types of CTC assays
  - Rapid and straightforward work flow
  - High sensitivity
- Giant CAMLs
  - More prevalent than CTCs
  - Many potential clinical applications

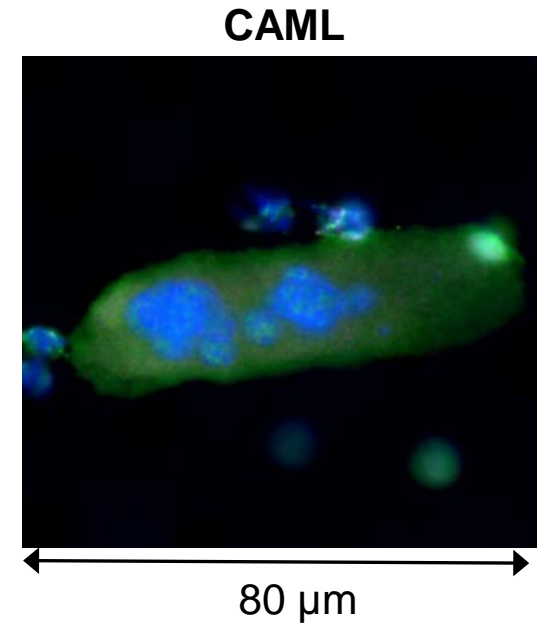

# Thank you

## ALL THE PATIENTS

Collaborators	Research Institute
Stuart Martin, Ph.D. Saranya Chumsri, M.D. Monica Charpentier, M.D., Ph.D.	University of Maryland Baltimore
Massimo Cristofanilli, M.D. FACP	Thomas Jefferson University
R. Katherine Alpaugh, Ph.D.	Fox Chase Cancer Center
Raymond C. Bergan, M.D. William J. Catalona, M.D.	Northwestern University
Thai Ho, M.D., Ph.D.	Mayo Clinic
Steven Lin, M.D., Ph.D.	MD Anderson
Susan Tsai, M.D.	Medical College of Wisconsin
Martin Edelman, M.D., FACP	University of New Mexico
Christian C. Haudenschild, M.D.	George Washington University



# Thank you

**Cha-Mei Tang, Sc.D.**

President and CEO

(office) 301-983-1650 (cell) 240-441-3311

[cmtang@creatvmicrotech.com](mailto:cmtang@creatvmicrotech.com)

[www.creatvmicrotech.com](http://www.creatvmicrotech.com)